

# CuddlePea®

## Bonding Doll

### Providing the comforting scent of Mom when skin-to-skin contact and breastfeeding are not possible.

CuddlePea readily absorbs the mother's scent with contact directly against the skin. When CuddlePea is placed with baby, the familiar scent of Mom helps to continue the bonding process started in the womb.

CuddlePea provides the "scent-connection" until skin-to-skin contact and breastfeeding are possible.

### Instructions For Use

#### In The Hospital

CuddlePea readily absorbs Mom's scent. Cuddle with CuddlePea bonding doll directly against your skin for 12 hours or longer, so it absorbs and retains your scent. Then, while holding your baby, introduce CuddlePea, or place it near your baby in the incubator, under your caregiver's supervision. The scent will help your baby associate CuddlePea with you, providing a sense of familiarity and comfort.

#### At Home

Follow the instructions above. Consider using CuddlePea to help soothe your baby when in a stroller, car seat, at daycare or whenever you can't be right next to your little one. Never leave baby unattended. REMOVE CuddlePea and all plush toys, stuffed animals, and decorative items at sleep or nap time. Nothing but the baby in the crib, according to American Academy of Pediatrics and the US Consumer Product Safety Commission guidelines.

### Product Specifications

Phthalate-free, latex-free, BPA-free and DEHP-free.

100% cotton fabric with polyester fiber fill (in head and arms).

For single baby use only.

Meets US Consumer Product Safety Commission guidelines.

Meets ASTM F963-17 Standard Consumer Safety Specification for Toy Safety.

Dimensions: 11" x 8.25".

Weight: 0.05 oz.

For babies 0 months +.



Use only under adult supervision.

### Cleaning

Spot clean with warm water or hand wash. Do not use bleach. Air dry.

After washing, cuddle and sleep with CuddlePea again so it reabsorbs your scent.

3-1902-001A